

Size Information

Universal

Size	Neck circumference	Height
	30-45 cm	11-17.5cm

Washing Instruction

Washing in cold water, gentle cycle and a mild soap.

Do not dry clean or tumble dry.

Do not use bleach.

Dry in an air current, and do not expose this brace to any heat source.

Caution

If allergic skin rashes occur, please consult your physicians.

Do not use more than 24 hours of continuous.

Indication

Cervical strain and sprain
Cervical spondylitis and spondylosis
Anterior discectomy
Anterior cervical fusion
Whiplash injury

Contraindication

Paraesthesia
Severe varicose vein
Skin disorders or injuries in the relevant area
Impaired lymphatic drainage

Recommendation

These products are to be used only by those which have been medical specialist. In order to dispose of the product, comply with the legal norms of your country. If any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Classification

Class I medical device, non sterile, no measurement function. This product fulfils the Medical Device Regulation (EU) 2017/745.



Application Instruction

Cortex Cervical Collar




INSTRUCTIONS FOR USE


CORTEX-80




1.Place posterior cervical panel of Armour Cervical Collar at the back of cervical spine.

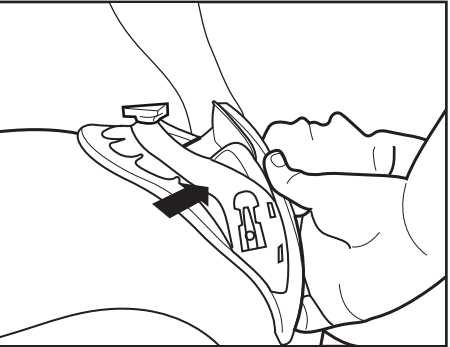


2. Place anterior cervical panel at the front side
The front panel should be at the lowest setting to begin, then plug both buckles.

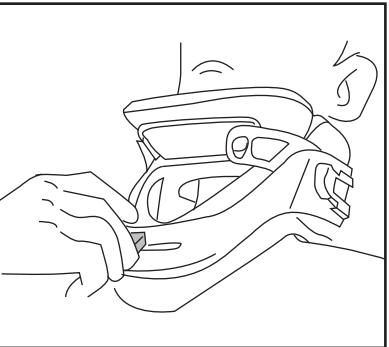

3.To adjust height of Armour Cervical Collar. Pull out the lock knob and slide to the right to increase the height of collar, and to the left to decrease the height. Push back the knob when the collar is at appropriate height.

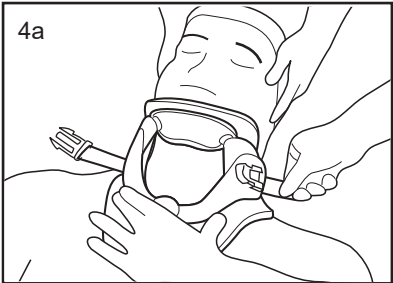

4a

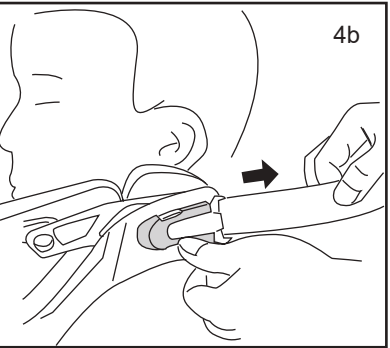

4b

4a. Then plug both buckles
4b. Adjust the strap to appropriate tightness.

5.Wear and remove the collar by releasing and plugging the buckles, once the tightness and height have been adjusted appropriately.