

Instructions for Users

Model: KS-97

KNEE SLEEVE

Knee Suspension Sleeve (KS-97) is intended for use with hinged knee braces to provide additional support and comfort and to help prevent brace migration or skin irritation. The brace is effective in providing soft tissue compression to help aid the healing process. It may be applied in various ways to provide support, relief or comfort.



1.1 Apply UNDER another brace. This application is intended to prevent slippage of the outer knee brace or to provide additional comfort or padding. Sleeve may be applied above the knee, at the knee or under the knee as needed.



1.2 While in a seated position, apply sleeve directly to thigh, or directly over the knee or to the upper calf as prescribed by your physician. Pull the handle firmly for a snug fit and secure in place. DO NOT OVERTIGHTEN.



2.1 Apply OVER another brace. The sleeve may be applied over another knee brace to help secure or stabilize the brace.



2.2 While in a seated position, apply the sleeve by wrapping around the brace on upper, middle or lower section of the inner brace and securing for a snug fit. DO NOT OVERTIGHTEN.



3. The handle may be detached from the strap for cleaning. When reattaching, make sure the holes extend beyond the end of the sleeve and the labels are on the opposite side.



CARE AND CLEANING INSTRUCTIONS

Hand wash with a mild soap and let air dry.

KNEE SLEEVE SIZE CHART

Size	Thigh Circumference	Calf Circumference
Universal	One size fits most	