

Instructions for Users

Model: VECTOR

VECTOR OA KNEE BRACE

VECTOR OA Knee Brace is intended for the support of Uni-compartmental osteoarthritis of the knee, rheumatoid arthritis, chondromalacia patella, degenerative meniscal tears, and collateral ligament protection.



1. Carefully cut off the attached plastic bag with the extension and flexion parts inside. Open all the buckles and extend all the fastener straps. Remove the snaps to the buckles from each of the straps, and lock them into the corresponding buckle. Make sure they are locked into place.




2. Position the knee brace on your leg so that the hinge is centered to your knee. Bring the strap under your leg and thread it through the rectangular slot at the end of the buckle. Next, pull the strap in place against itself. Adjust the length of the strap or cut off the excess strap if needed. Do this for each of the straps from No.1 till No.4.



3. Once your knee brace is properly aligned to your knee cap, stand up and with a slight bend to your knee, tighten all the straps in place so that you can be assured that the brace will not slide up or down or rotate on your leg. DO NOT OVERTIGHTEN.



4. Hinge Adjustment: Using the included Allen Wrench, loosen both screws on the hinge cover. Place the paired hinge stops provided. Place one part of the pair on each side of the hinge, to restrict the range of motion of the hinge bend.

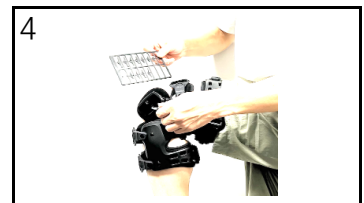

5. Unloading Adjustment: Using the included Allen Wrench, rotate the unloading dial located above the ROM hinge. Turn it clockwise or counterclockwise to adjust hinge to desired angle.



CARE AND CLEANING INSTRUCTIONS

Hand wash with a mild soap and let air dry.

OA RELIEVER SIZE CHART

Size	Thigh Circumference	Calf Circumference
Universal	One size fits most (Left/Right)	