

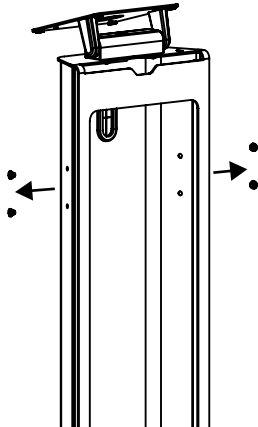
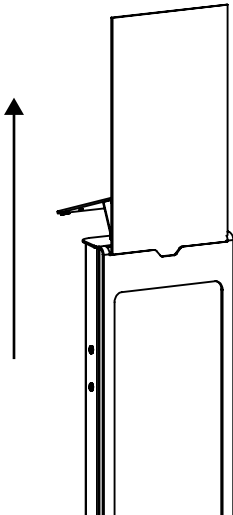
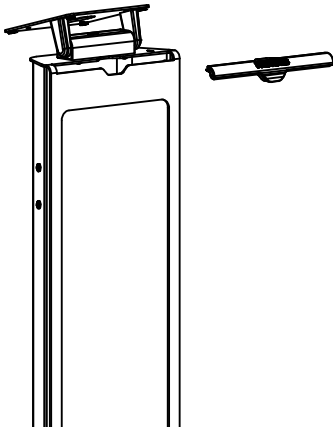
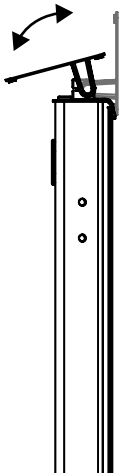
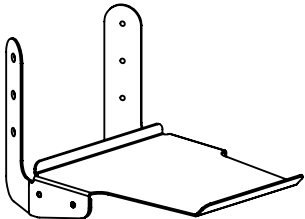
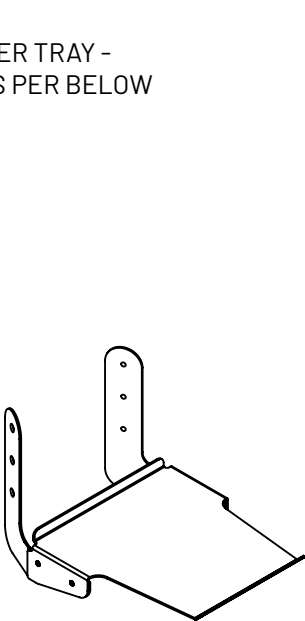


- 1
- TILT VESA MOUNTING PLATE
- 2
- REMOVE RUBBER CAP
- 3
- REMOVE COVER PANEL
- 4
- REMOVE RUBBER CAPS

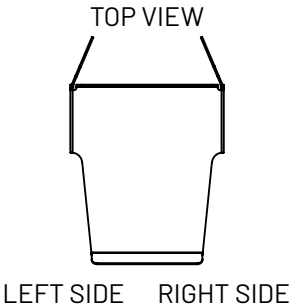


- 5
- ASSEMBLE WINGS AS SHOWN - L TO L AND R TO R
USING 4 NUTS (PROVIDED)
- 6
- TURN PRINTER TRAY -
ENSURE IT IS AS PER BELOW
- 7
- SLIDE PRINTER TRAY ONTO
THE BRANDME STAND

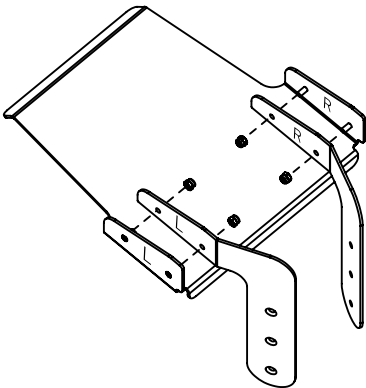


8

SELECT FROM HEIGHT OPTIONS

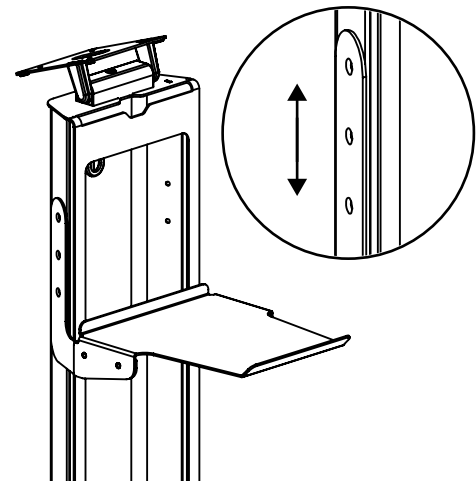

9


TIGHTEN 4 SCREWS AND NUTS (PROVIDED) - 2 PER SIDE

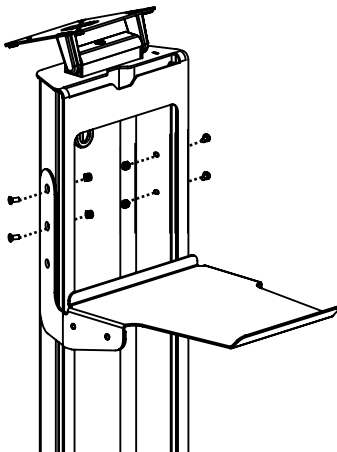

10

OPTIONAL CABLE HOLDERS

OPTIONAL ADHESIVE TAPE

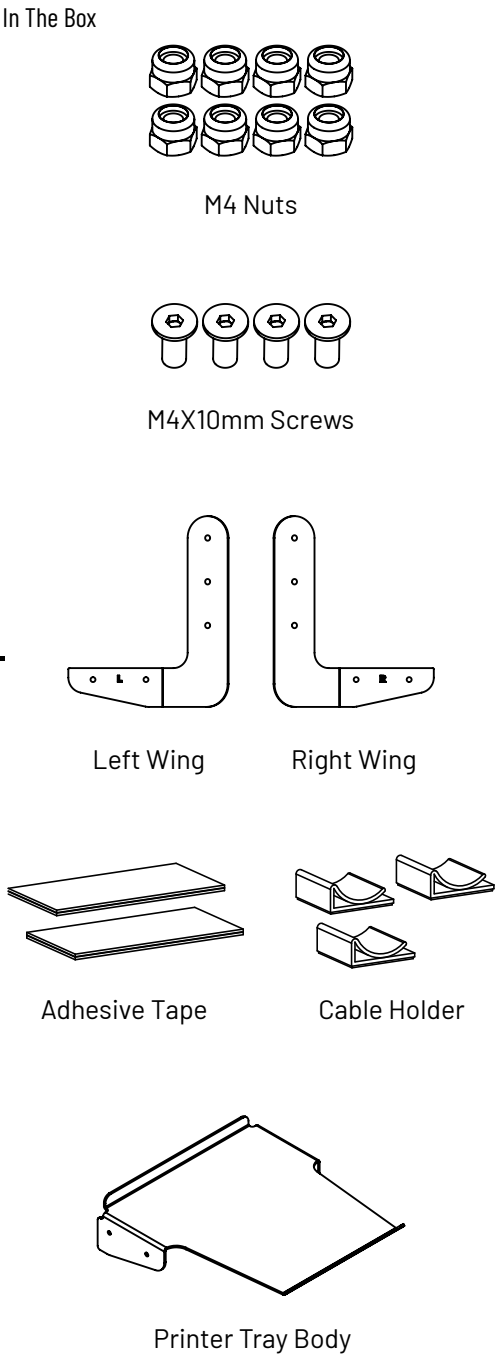

11

REPLACE COVER PANEL AND THEN REPLACE RUBBER CAP

