

CASSETTE HOLDERS

INSTRUCTIONS FOR USE

COPYRIGHT ©2026. ALL RIGHTS RESERVED



USE

Cassette holders are designed to securely position compatible imaging receptors during radiographic procedures.

Before each use, inspect the holder and ensure that all clamps, adjustment mechanisms, locking components, and casters, when applicable, are secure and functioning properly. Always make sure the imaging receptor is properly secured before positioning or beginning a procedure.

Use all adjustment and locking mechanisms as intended, and do not exceed the size or weight capacity specified for the cassette holder.

MAINTENANCE

Regularly inspect the cassette holder for signs of wear, loose components, or damage. Check clamps, knobs, adjustment mechanisms, moving parts, and casters for proper operation.

Keep moving and locking components free of dirt and debris that could interfere with their operation. Do not use the cassette holder if any component is damaged, loose, or unable to securely support the imaging receptor.

STORAGE

Store the cassette holder in a clean, dry indoor environment when not in use. Keep the holder on a stable, level surface and lock the casters, when applicable, to prevent unintended movement.

Avoid prolonged exposure to excessive moisture, heat, or corrosive substances. Store the holder where it will not be struck or damaged by other equipment.

WARRANTY

Phillips Safety Cassette Holders are covered by a one-year warranty against manufacturing defects.

Attention: The warranty does **not** cover normal wear and tear, misuse, or damage caused by overloading the holder or its support points.

CLEANING

Remove the imaging receptor before cleaning. Wipe the holder with a soft cloth dampened with mild soap or detergent and water. Pay particular attention to clamps and other contact surfaces where dirt or residue may accumulate.

Do not use abrasive cleaning materials or saturate joints, adjustment mechanisms, or other moving components. Wipe away any remaining cleaning solution and dry all surfaces thoroughly before use.

IMPORTANT

Always follow your facility's established cleaning, disinfection, and infection-control procedures. Before using any disinfectant or chemical cleaner, verify that it is compatible with the cassette holder's materials and finishes.