



Intraoperative Patient Eye Protection Shield

Assists in protecting patient eyes from unintended injury, instruments, debris and spills under general anesthesia.

Intended Use

- SUPINE (Face Up) General anesthesia only with controlled airway: GETA or LMA Anesthesia
- Device is not intended for use in an awake patient.
- Non-Sterile. Single use only. Do not re-use.
- Useful in DaVinci, Laparoscopic, Breast, and Beach Chair Shoulder Procedures.

Warnings, Precautions and Contraindications

- Do NOT use in Prone or Decubitus position.
- Store out of direct light and sunlight until used. Padding discolors with extended UV or sunlight.
- Do not use if packaging or device is damaged, ripped, punctured, or deformed. Inspect before using.
- Do not use if padding has separated.

Ensure and Maintain Proper Fit

- Lay on face, and over the eyes, resting on forehead and chin.
- Secure with light tension, using enclosed strap or anesthesia harness.
- Keep padding away from the eyes.
- Periodically check position and re-adjust if necessary.

USE OF THE SHIELD IS NOT A SUBSTITUTE FOR GOOD CLINICAL JUDGMENT AND SHOULD NOT ENCOURAGE THE OPERATOR TO BE LESS VIGILANT ABOUT EYE INJURIES.

Part #SE 5585-2017

DOMA

SAVEGEYES®



Intraoperative Patient Eye Protection Shield

Operator Instructions



1

Preparation:

- A** Patient under SUPINE GETA or LMA anesthesia.
- B** Check packaging.
- C** Remove Shield. Orient the Shield with the vertical cutout toward the patient's chin.

Pass the Shield over the airway tube and lay on patient's face, and over the eyes allowing it to rest on the forehead and chin. Keep foam padding away from eyes.



2

Attach tube to anesthesia circuit.



3

Secure the Shield with light tension using enclosed strap or anesthesia harness. Re-check fit to ensure breathing tube and Shield are in good position.



4

Prior to extubation, disconnect anesthesia circuit. Release strap, lift Shield off face, around GETA/LMA tube and discard.




5

Alternate Rapid Shield Removal

If rapid Shield removal is necessary:

Separate and hold padding open overlying chin and remove around breathing tube without disconnecting anesthesia circuit.

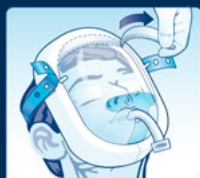

6

Remove Crescent-Shaped Tab

Patient Safety