NOTIONS DE BASE

OBJECTIFS

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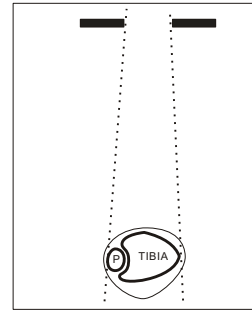
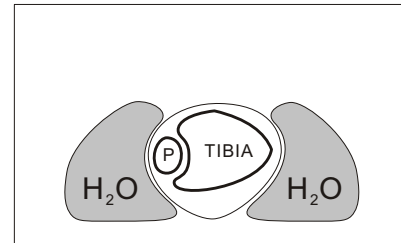
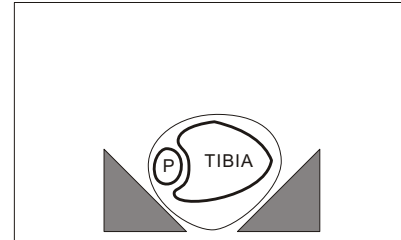
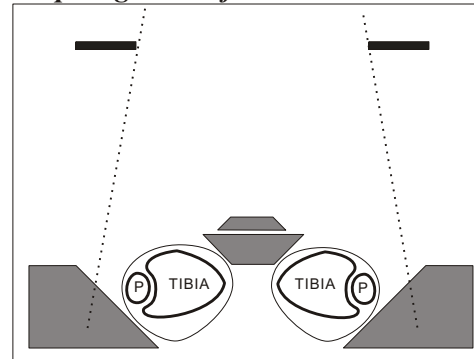
BASIC NOTIONS

OBJECTIVES

To even out the density of an image which would otherwise be too bright or too dark in some areas. This saves films, time and radiation. Also to immobilize at the same time.

MEANS

1. Closure of the **diaphragms** inside the part examined.
2. Use of a **bolus** near the part to be radiographed. This cuts the peripheral "black" which spreads over the anatomy, but also casts artifacts.
3. Use of a **compensating filter**:
 - a) a "black" area will be attenuated, such as the periphery of the shoulder with the "BOOMERANG";
 - b) a "white" area may be better penetrated while protecting the rest of the film, v.g. C-7 with the "INGOT".
4. Local **digital** adjustments on areas too bright or too dark, but not simultaneously with normal areas.
5. An **electronic threshold** cutting off overexposed areas of the image.
6. A **combination** of many factors.

MATERIAL AND ABSORPTION

1. **Rice, flour, etc.** do not absorb sufficiently.
2. **Water** and **serum bags** would be good mainly as boluses, but not practical. Near the trunk they would be large and heavy, although adding an opaque substance improves them.

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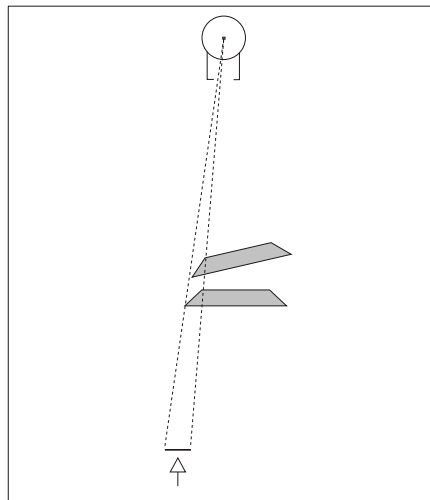
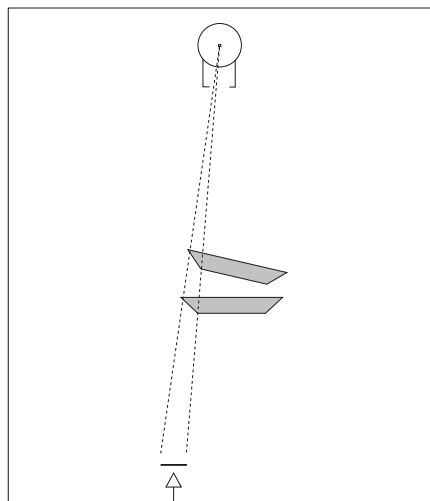
Gradient estompé.

Soft gradient.

3. Rigid **plastic** is good, but the tapered edge may break, thereby injuring a patient or a technician; it does not adhere vertically.

4. Metals, such as **lead** and leaded plastics, are generally too dense. **Copper** and **aluminum** are useful.

5. **OCTOSTOP® compensating filters** are made of **silicone rubber**. Their absorption is ideal at twice that of muscle, which permits placing them on only one side of the patient.

They are light to carry, easy to handle, readily identifiable and not cumbersome. They are positioned with precision close to the patient. If dropped, they do not break or harm anyone. In the vertical position, they adhere with very little pressure (wash with water if they slip). If too sticky, interpose tissue paper.

They resist alcohol and heat. They are waterproof.

6. PHARAOH boluses in polyurethane surround the head and the neck completely: they absorb like water.

GRADIENT OF ABSORPTION

It is proportional to the slope of the wedge: the steeper the gradient, the sharper and more obvious is the absorption. The more gradual it is, the less perceptible is its shadow. It is measured in degrees, or as a fraction:

BOOMERANG: 30°-45° or 1/1.7-1/1

INGOT: 45° " 1/1

GENTLE SLOPE 8° " 1/7

PRISM: 37° and 52° " 1/1.3 and 1/1.8

In addition, the gradient varies according to the orientation of the filter. It increases as the axis of the wedge becomes more parallel to the direction of the X-ray beam; it decreases in the reverse situation(see figures).

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
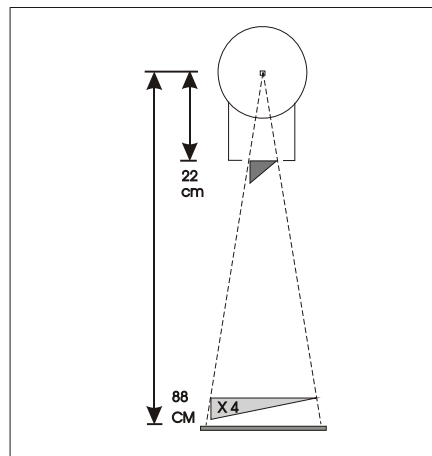
Gradient at collimator vs on patient

Bord effilé.

Tapered edge.

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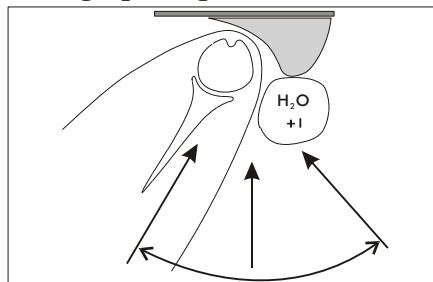Special tomography.

POSITION

1. *Behind the patient.*
2. *In front of the patient:* saves radiation to the patient.
3. *At the collimator*, with Velcro[®] as an inexpensive means of attachment. Here the gradient diminishes by a factor of 4, since its width is magnified in the proportion of 88:22 cm (see diagram).

SHAPE

1. *Curved at 90° for the BOOMERANG.*
2. *Rectilinear for other filters.*
3. *Adapted.*

TAPERED EDGE

It is fragile and must not be stretched or torn off with fingernails. It is nevertheless long lasting with reasonable care. Do not fold over, as this may create an artifact.

SPECIAL TOMOGRAPHY (axial of the shoulder)

When a superficial region is **very thick**, controlling its density is difficult: because the wide sweep of very "black" air coming from levels far from the cut, largely overlaps the anatomy, which then becomes much too dense. Nevertheless, two solutions are available:

- A. *Place a filter against the patient and add a bolus containing iodine.*
- B. *And/or use a highly absorbing filter at the collimator.*

STERILIZATION and DISINFECTION

Autoclave is recommended for sterilization; alcohol or Zephiran[®] for disinfection.

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THE RIDGES PLUS

The RIDGES PLUS insure constant optimum density of digital arteriography (and venography) of the lower extremities down to the feet. By removing halation, they facilitate digital subtraction. With the cushions, they are an excellent immobilizer. See the video included with each kit (also available for free).

DESCRIPTION (Figure 1)

They are all thinner distally. Two incorporated midline opaque rulers allow measurements of site, length and caliber of lesions. The **inter-thighs filter (A)** has a pyramidal shape. Protect it with thin wrapping. The two **inter-legs filters (B)** are bilobed; the main one is thicker. The two **external ridges (C)** contour the calves and the dorsum of the feet; they also suit the upper extremity. A **triangular (D)** filter borders the sole of the feet. Extensions may cap midline filters for longer legs.

PREPARATION (Figure 2)

Place the two Velcro® bands on the table, felt facing upward. Lay the main inter-legs upside down, then the accessory one over, with all arrows in the direction of the arterial flow. Place a tape with the adhesive side upward at the level of the heels.

INSTALLATION (Figure 3)

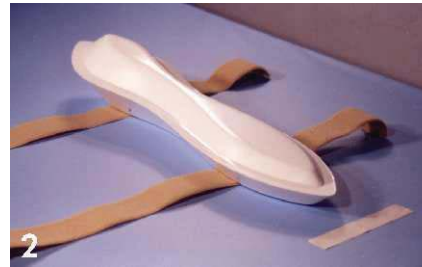
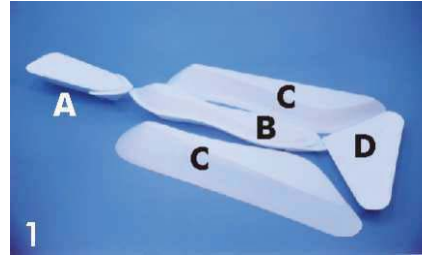
The patient lays with his legs on the bands, while the inter-legs filters are re-positioned over them, the depressions against the calves. Stick the heels together with the adhesive tape underneath. Then tighten the straps around both filters and legs. If the knees are difficult to bring together, flex them slightly. Lean the lateral ridges against the legs and the feet. Push the triangle filter upside down against the sole of the feet. The inter-thighs filter will be placed before or after the puncture, with the top downward and the pointed end toward the knees.

IMMOBILIZATION (Figure 4)

Adhesive tapes are anchored to the sides of the table. Protect the cushions with a thin plastic sheet. A loop of adhesive tape draws each foot in external rotation against a prism-shaped cushion. Crossed adhesive tapes press down the pyramidal filter between the feet, preventing dorsal flexion and rotation. Finally, a transverse adhesive presses the upper part of the inter-legs filters between the knees that are squeezed between the lateral filters.

VERIFICATION

Under fluoroscopy or with films, make sure that all is centered and symmetrical; that knees and heels are close together; and that there is no halation.

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IMPORTANT PRECAUTIONS

1. Read this educational brochure attentively.
2. Do not filter when there is little or nothing to be gained.
At the beginning, conduct your own experiments.
3. In general, adjust technical factors for the unfiltered area.
4. If an artefact is observed, indicate it on the film;
or better, take another corrected exposure.
5. Observe local practices. Consult your radiologist.
6. Send us your suggestions.

IMPORTANT

Must be used only by qualified personnel and under the direct supervision and responsibility of a physician. When applying traction, caution should be exercised when the following conditions are present or possible: fractures of the pelvis; spine or upper extremities; uro-genital or abdominal trauma; or all other circumstances that might be contraindicated.

OCTOSTOP[®] Inc. and its personnel do not accept any liability for how its products are used, for the indications, consequences or any situations directly or indirectly related to its products.