


ASSEMBLY INSTRUCTIONS

PIG[®] Surgical Absorbent Mat Roll Dispenser



Step 1 – Cross bases to form an “X” with wheels facing up.

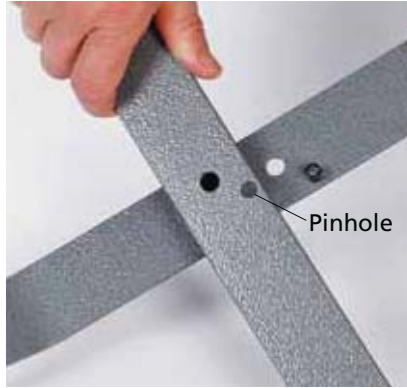

Step 2 – Align pin with pinhole to hold bases in position.



Step 3 – Insert bolt through both bases. Threads should be pointing up when base is upright.



Step 4 – Screw either end of pole onto threaded end of bolt.



Step 5 – Slide friction disc over pole so it rests on the base.



Step 6 – Screw connector onto top of pole for a finished look.



Step 7 – Load mat onto pole so it rests on the friction disc. Your PIG[®] Mat is ready to roll!!