



CLASSIC SERIES TREATMENT TABLES


MODELS 1010, 1020 1030, 1013, 1015,
1025, 1035 & 1135

with ADJUSTABLE BACKREST

& MODELS: 100, 200, 300, 1300,

1015-088,1025-088, 1035-088 & 1135-088

with NON-ADJUSTABLE, FLAT TOP

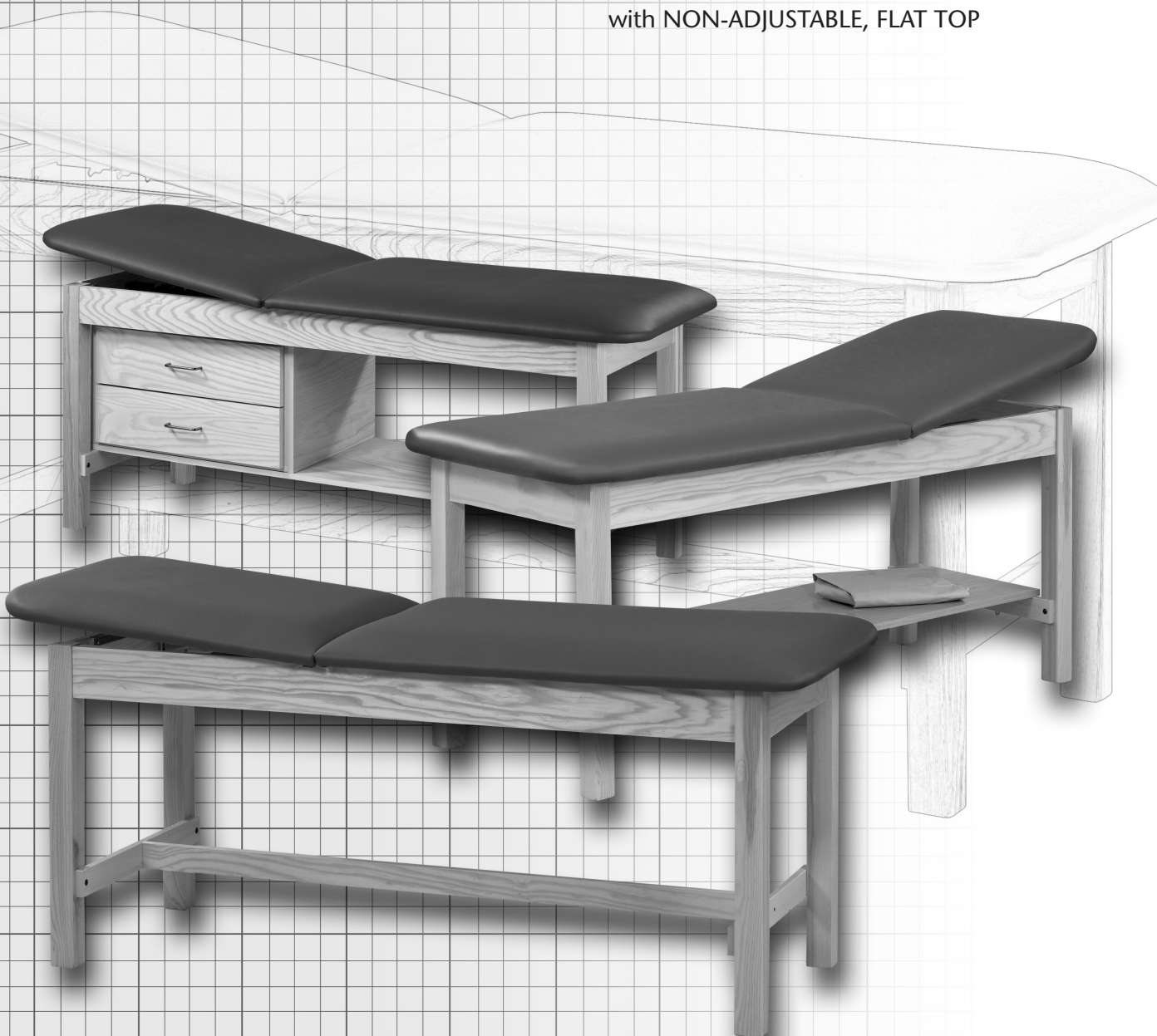

ASSEMBLY INSTRUCTIONS

ASSEMBLY INSTRUCTIONS FOR CLASSIC SERIES TREATMENT TABLES

MODELS: 1010, 1020, 1030, 1013, 1015, 1025, 1035 & 1135 with ADJUSTABLE BACKREST & Tables with pre-attached backrest

MODELS: 100, 200, 300, 1300, 1015-088, 1025-088, 1035-088 & 1135-088 with NON-ADJUSTABLE, FLAT TOP

& PRE-ATTACHED TOPS begin assembly with STEP 4

Thank you for counting on Clinton to meet your medical equipment needs by purchasing this fine equipment. All Clinton equipment is designed and manufactured to provide years of dependable service because we want you as a customer now and in the future. Below are easy to follow instructions on how to properly assemble your equipment. Please contact us at 800-441-9131 if you experience any difficulty with the assembly or have questions.

INTENDED USE

Clinton treatment tables are specifically designed and engineered for the use by a single person at a time, who is attended to and observed by trained medical personnel. Never leave anyone on the table unattended. Person's weight and the combination of force exerted on the table during treatment is not to exceed **400 lbs. for models: 1010, 1020, 1030, 1013, 100, 200, 300 & 1300, and 500 lbs for models: 1015, 1025, 1035, 1135, 1015-088, 1025-088, 1035-088 & 1135-088**

KD Quantity	Hardware Package	Pre-Attached Quantity
(4)	1 1/2" Truss Head Screws	(4)
(8)	1" Truss Head Screws	(0)
(8)	5/16" Washers	(4)
(4)	5/16" Nuts	(4)
(2)	2 1/4" Pan Head Screws	(2)
(2)	3/4" Barrel Nuts	(2)
(4)	1 1/4" Truss Head Screws (1030 & 1013)	(4)

TOOLS REQUIRED

Phillips Screwdriver and Adjustable Wrench

IF TOP IS ATTACHED TO FRAME, SKIP TO STEP 4

1

Place the upholstered sections on a flat surface with the foot end on the left and fabric tag toward the center of the table as shown. Place the head end on right with hinge toward the center of the table. Place frame on top with channel, teeth down, above the head end.

FOR MODELS WITH ADJUSTABLE BACKREST



fabric tag


center hinge

teeth

2

Align the holes in the hinge with the holes in the center board and fasten to center board using 4 silver, 1" truss head screws.

FOR MODELS WITH ADJUSTABLE BACKREST

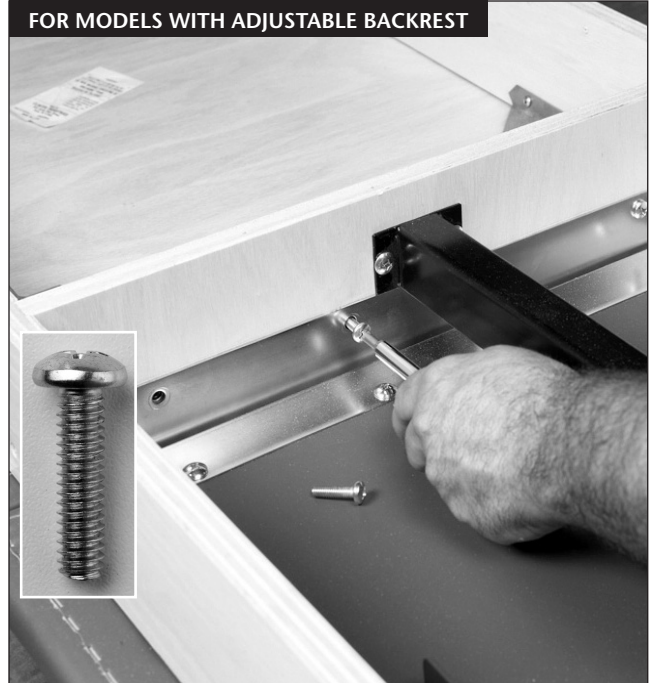


3

Finger tighten the top's foot end to frame using 4 silver, 1" truss head screws and 5/16" washers.

Slide the foot end toward the center board and finish tightening screws.

FOR MODELS WITH ADJUSTABLE BACKREST

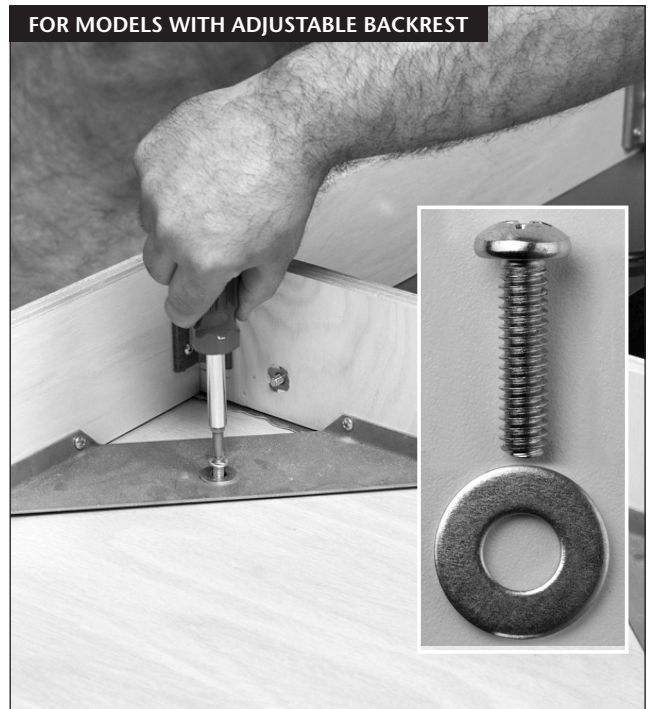

4

Position leg in the leg brace with H- brace hole facing inward. Attach the leg using $\frac{5}{16}$ " washer and $\frac{5}{16}$ " nut. Finger tighten nut now, and fully tighten after step 6 is completed. Repeat with remaining legs.



5

Position the H-braces with finished edge toward top with the cut-out slot facing toward the inside of the table and attach using the four (4) $1\frac{1}{2}$ " truss head screws.



6

Insert the barrel nuts into the holes at each end of the spanner. Make sure the threaded hole in the nut is position to accept the pan head screw that will be inserted through the H-brace.




7

Insert the $2\frac{1}{4}$ " pan head bolts through each H-brace and into the barrel nuts in the spanner. Finger tighten until the screw catches the threads in the barrel nut, then secure with a screwdriver.



8


Return to each corner leg and finish tightening each nut. **DO NOT OVER TIGHTEN.**

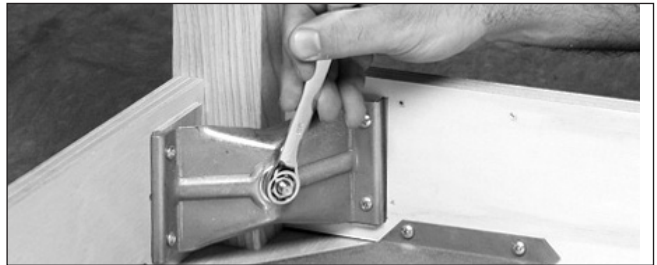

9

SHELF ASSEMBLY

Invert the table to an upright position on all four legs, using caution when inverting the end with the adjustable headrest. Apply 4 adhesive backed hook strips to top edge of H-braces. Place the full-length shelf on the spanner with the loop strip underside over the hook strips on the braces.

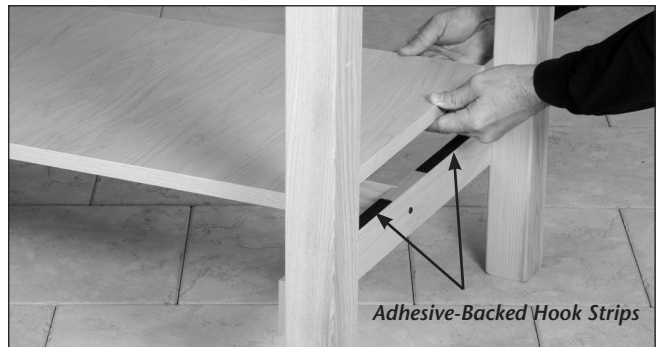

FOR SAFETY PURPOSES, TAKE TIME TO CHECK ALL SCREWS, BOLTS AND NUTS TO INSURE THAT THEY ARE SECURELY TIGHTENED AND THAT THE TABLE HAS BEEN PROPERLY ASSEMBLED ACCORDING TO THESE INSTRUCTIONS.


PROCEED TO STEPS 9 & 10 ON BACK COVER FOR MODELS WITH TIERED SHELF OR DRAWERS.

10**1030, 1013, 300, 1300, 1035, 1035-088, 1135 & 1135-088 SHELF/DRAWER ASSEMBLY**

Using assistance, place the drawer box (with the drawers removed) or tier shelf on the full shelf, aligning it with the end of the shelf that has the pre-drilled holes.

**11****1030, 1013, 300, 1300, 1035, 1035-088, 1135 & 1135-088 SHELF/DRAWER ASSEMBLY**

Align pre-drilled holes and edges of the drawer or tier shelf section. Fasten shelf by screwing the supplied 1 1/4" black truss head screws through shelf and into pre-inserted, threaded inserts.

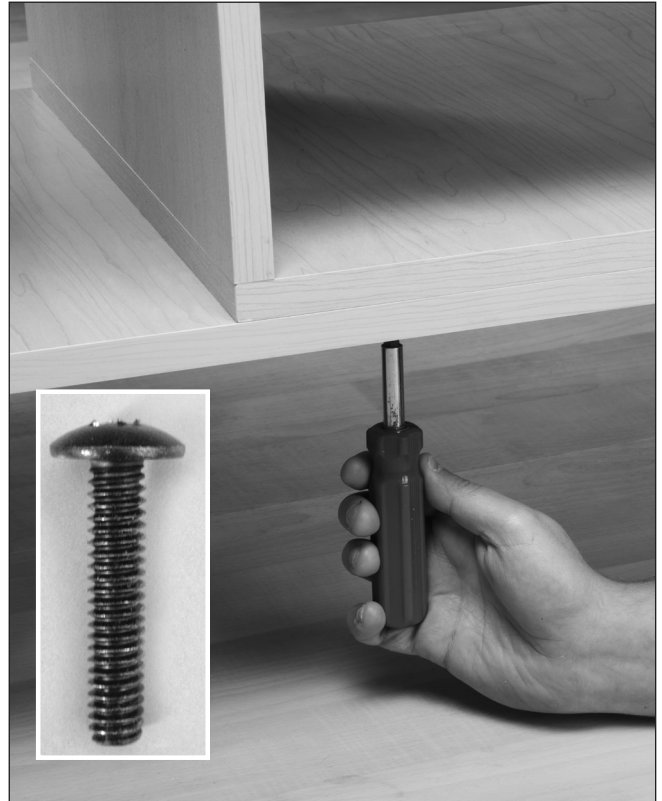

CARE & CLEANING

Clinton products are built to be durable. With proper care, your Clinton equipment will provide years of superior looks and performance. Clean table with a soft cloth and any mild cleaning agent that may be required for sanitation. NEVER use abrasive scrubbing agents that may scratch the surfaces. **Before using any cleaning agents, try them on a small inconspicuous area to insure that they will not harm the upholstery or any other surface. Do not use alcohol based cleaning agents.**

Periodically check that all fasteners are secure and operating properly.

