

Radiology Table Pads

CLEANING & CARE INSTRUCTIONS

Thank you for choosing our pad for your radiology accessory needs. To ensure the longevity of your new table pad, we recommend the following guidelines for cleaning and care.

Cleaning & Care:

Do:

- It is recommended to clean with cold water and a mild detergent immediately after contact with fluids. Household 409 Spray can be used to clean the surface fabric or vinyl and remove dust and dirt from your table pad.
- To disinfect your table pad, it is recommended to use disinfectant wipes safe for use on vinyl materials. Allow a drying time of at least 3 minutes.
- Store your table pad on a pad rack when not in use.

Don't:

- Do not use Bleach or any harsh chemicals on your table pad.
- Do not autoclave or launder.
- Do not soak/submerge table pad in water.

Failure to adhere to the above recommendations will shorten the life of your table pad, promote cracking, and will possibly void your warranty.