

# Stealth-Cote Sponges

## CLEANING & CARE INSTRUCTIONS

### Cleaning & Care:

We all know that in a medical environment, it is easy for surfaces to become dirty and soiled from blood, bodily fluids, betadine, barium, and other fluids.

#### Do:

- Clean with cold water and a mild detergent immediately after contact with fluids. Household 409 Spray can be used to clean and remove dust and dirt from your coated sponge.
- To disinfect your sponges, we recommend the use of disinfectant wipes. Allow a drying time of at least 3 minutes.
- Store your sponges in a clean and dry area when not in use.

#### Don't:

- Do not use Bleach or any harsh chemicals on your coated sponge.
- Do not autoclave or launder.
- Do not soak/submerge sponges in water.

Failure to adhere to the above recommendations will shorten the life of your coated sponges, promote discoloration of the surface, and will possibly void your warranty. Techno-Aide provides a one year warranty on workmanship and materials.