

BOLSTER SPONGES

CLEANING & CARE INSTRUCTIONS

Do:

- Clean with cold water and a mild detergent immediately after contact with fluids. Household 409 Spray can be used to clean and remove dust and dirt from your coated sponge.
- To disinfect your sponges, use Alcohol Free disinfectant wipes. Allow a drying time of at least 3 minutes or as advised by the wipe manufacturer.
- Store your sponges in a clean and dry area when not in use.

Don't:

- Do not autoclave
- Do not launder
- Do not soak or submerge sponges in water