

INSTRUCTIONS FOR USE



Allen® Patient Transfer Board™ A-83000, A-83001, A-83005



Warnings


Prior to using this or any other type of medical apparatus with a patient, it is recommended that you read the *Instructions for Use* and familiarize yourself with the product.







INSTRUCTIONS FOR USE

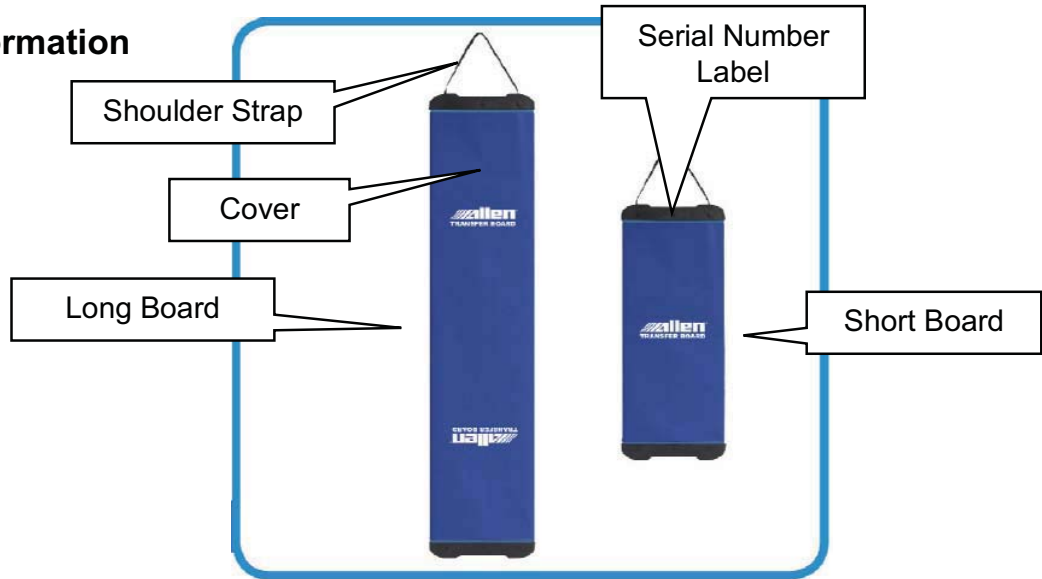

Safety Information



-  Do not use if product shows visible damage
-  Do not exceed 500 lb patient weight limit
-  Ensure both the OR Table / Hospital Bed and the stretcher are locked and placed next to each other with as little gap as possible. Do not use the transfer board if the gap is larger than 4 inches (10.2 cm) . Ensure the receiving surface is slightly lower than the adjacent surface to assist with the transfer.
-  Do not use the carrying strap to pull during the transfer of a patient.



INSTRUCTIONS FOR USE



Setup and Use

- 1 Use shoulder strap to transport device.



- 2 Beds have wheels locked and are placed as close together as possible.

Do not use transfer board if gap is larger than 4 inches (10.2 cm).

Ensure the receiving surface is slightly lower to assist with the transfer.



- 3 If desired, cover the Transfer Board with a plastic disposable cover to protect it from contamination.

Ensure both ends of the Transfer Board are protected by the cover.



INSTRUCTIONS FOR USE




Setup and Use

- 4** Use a draw sheet to logroll the patient and place the board under the patient.
- With the long board, ensure the patient's feet and head are on the board. Always have one clinician supporting the head throughout the entire transfer.
- For tall patients, place a pillow on the blue cover under the feet to assist with transfer.
- Ensure a clinician is supporting the patient's feet when using the short board.



- 5** To transfer, one clinician should raise the patient using the draw sheet to ensure the patient's weight is on the board. Another clinician will pull on the draw sheet from the other side to complete the transfer.

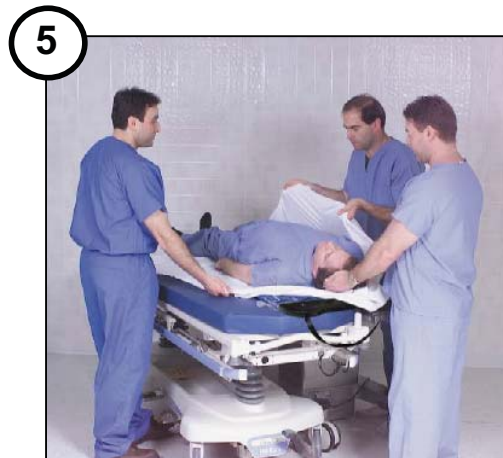

INSTRUCTIONS FOR USE



CLEANING AND MAINTENANCE

- After each use, clean the device.
- Clean and disinfect using a quaternary ammonium disinfecting/cleaning solution following the manufacturer's recommendation for achieving low-level disinfection.
- Read the cleaning product's directions and follow the instructions on the label. Use caution in areas where fluid migration may occur: *(example, if appropriate)*
- Wipe device with a clean, dry cloth.
- Make certain the product is dry prior to reinstalling to avoid damage.



CAUTION: Damage may result if product is cleaned with bleach.

All modifications, upgrades, or repairs must be performed by an authorized specialist

