



Child **CRiSis**™ Airway  
Management Trainer  
LF03633U  
INSTRUCTION MANUAL



5-Year Warranty

## ABOUT THE SIMULATOR

The **Life/form**® Child **CRiSis**™ Airway Management Trainer is the most realistic simulator available for the training of intubation and other airway management skills.

Nasco has taken great care to create an airway management trainer that is anatomically correct in respect to both size and detail. Landmarks include nostrils, teeth, tongue, oral and nasal pharynx, larynx, epiglottis, arytenoids, false and true vocal cords, trachea, esophagus, lungs, and stomach.

The **Life/form**® Child **CRiSis**™ Airway Management Trainer allows you to practice oral, digital, and nasal intubation, as well as suction techniques. The simulator was designed to use an uncuffed endotracheal tube measuring up to 5.0 mm I.D. With proper care, your **Life/form**® Child **CRiSis**™ Airway Management Trainer will provide years of valuable service.

## LIST OF COMPONENTS

- **Life/form**® Child Airway Management Trainer with Child **CRiSis**™ Torso

## SET UP

Your **Life/form**® Child **CRiSis**™ Airway Management Trainer is ready to use upon delivery. Simply remove from carton and unwrap packaging material.

## LUBRICATION

Before intubating the **Life/form**® Child **CRiSis**™ Airway Management Trainer, lubricate the simulator with the Nasco lubricant provided. (See figure 1)

## CLEANING THE FACE

Normal soil can be removed from the face of the trainer with mild soapy water. Nasco Cleaner (LF09919U) will remove stubborn stains. Simply apply Nasco Cleaner to soiled area and wipe clean with a soft cloth or paper towel. **Note:** Do not use Nasco Cleaner around the mouth and nostrils, as the residue of cleaner could be toxic.



Figure 1

Actual product may vary slightly from photo. Nasco reserves the right to change product color, materials, supplies, or function as needed.

## CAUTIONS

Nasco recommends the use of the enclosed aerosol lubricant or a similar vegetable-based lubricant for use with our **Life/form**® Child **CRiSis**™ Airway Management Trainer.

DO NOT use a silicone or similar lubricant. This will cause your simulator to dry and crack, and will automatically void Nasco's warranty on the trainer.


Never place the trainer on any kind of printed paper or plastic. These materials will transfer indelible stains. Ballpoint pens will also make indelible stains.

## AVAILABLE SUPPLIES

- LF03608U** Nasco Lubricant
- LF03627U** Replacement Lung Set
- LF03628U** Replacement Stomach
- LF03631U** Replacement Headskin with airway
- LF09919U** Nasco Cleaner

## ADD-ON COMPONENTS

- LF03611U** Defibrillation Chest Skin
- LF03612U** Left Arm — IV Training
- LF03613U** Right Arm — Blood Pressure Monitoring
- LF03614U** Right Leg — Intraosseous Infusion/Femoral Access
- LF03634U** Left Leg (Nonfunctioning)

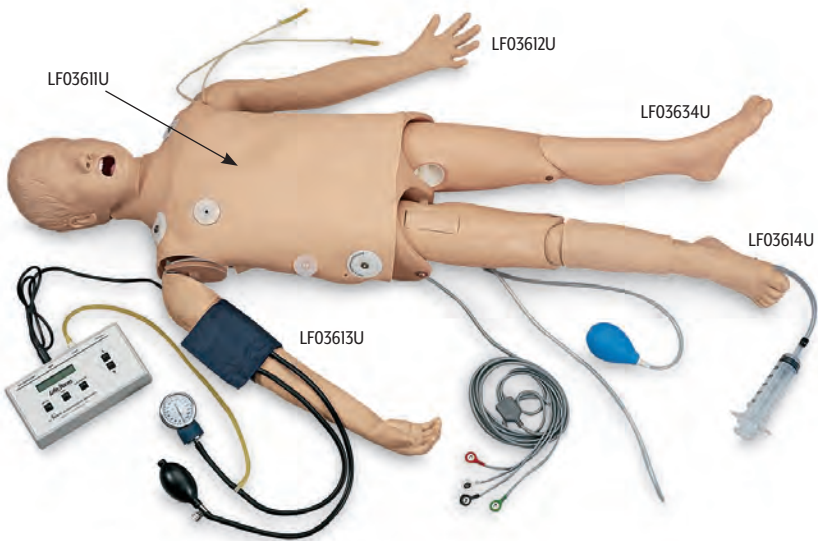

## Other Available *Life/form* Simulators

- LF03601U Adult Airway Management Trainer with Stand
- LF03602U Adult Airway Management Manikin
- LF03609U Child Airway Management Trainer with Stand
- LF03616U Child *CRiSis*™ Manikin
- LF03617U Deluxe Child *CRiSis*™ Manikin with Arrhythmia Tutor
- LF03623U Infant Airway Management Trainer with Stand
- LF03709U Infant *CRiSis*™ Manikin
- LF03953U *CRiSis*™ Manikin, Complete
- LF03955U Deluxe *CRiSis*™ Manikin
- LF03956U Deluxe "Plus" *CRiSis*™ Manikin
- LF03965U Adult *CRiSis*™ Auscultation Manikin
- LF03966U Adult *CRiSis*™ Auscultation Manikin with ECG Simulator



LF03609U



LF03623U



LF03709U

