


# Instruction Sheet

## Pass-Thru Cabinet #181785 or #181785T

- 1) Cut rough opening in wall 11 3/4" W x 12 1/8" H and construct inside wall framing of suitable material to accept cabinet mounting screws.
- 2) Slide the cabinet into the wall opening and secure it to the wall from inside the cabinet with (4) #8 screws. (not provided)
- 3) Before affixing the flanges to the wall, clean the surrounding area with Isopropyl alcohol.
- 4) Remove liner paper from blocks located at each corner of the flanges. Position each flange over the cabinet and up against the wall. Apply firm pressure at each corner to ensure a proper bond.

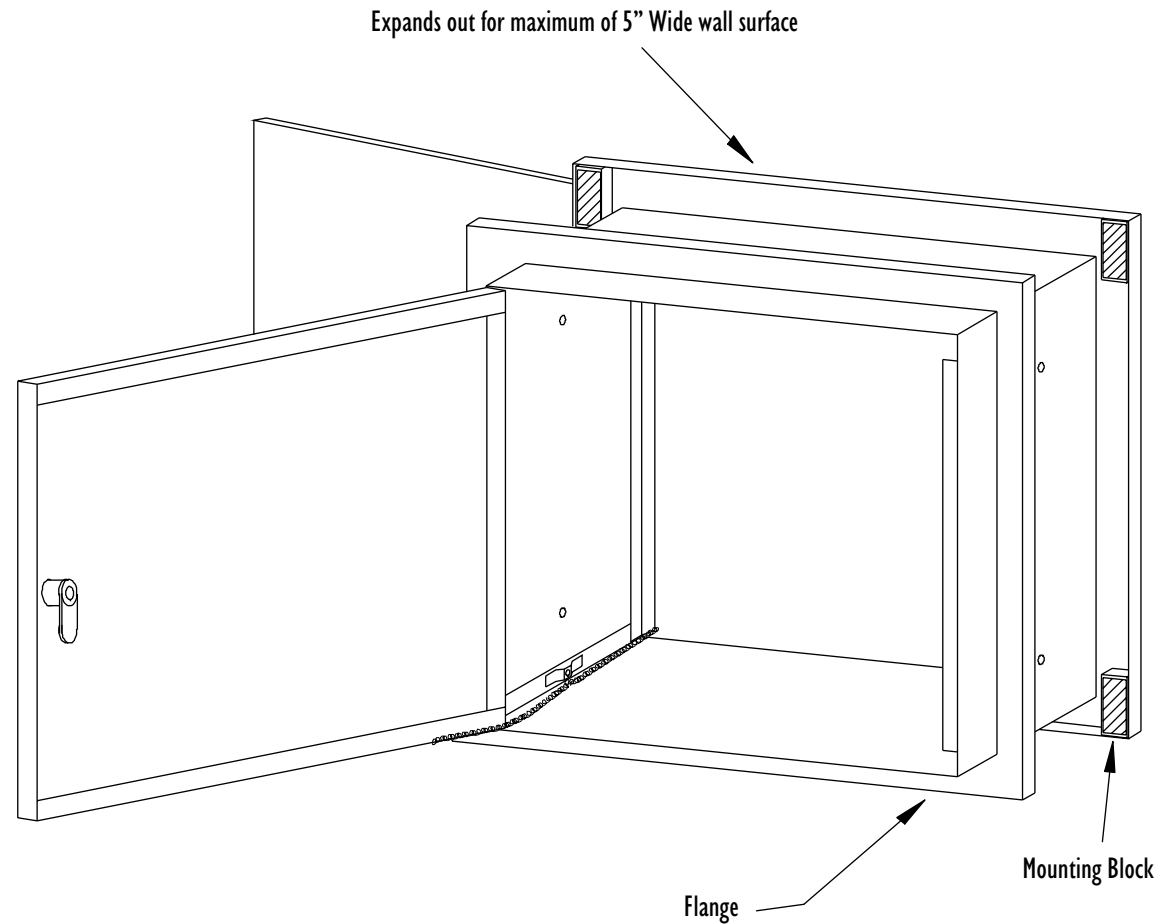

MANUFACTURERS OF **IBeam**® PRODUCTS  
**OMNIMED** INC.®