


Instructions For Use:

**Product Name: Multi-Task Armboard**

**Product Number: O-ABMT**



Prior to using this or any other type of medical apparatus, it is recommended that you read the Instructions for Use and familiarize yourself with the product prior to application on a patient.

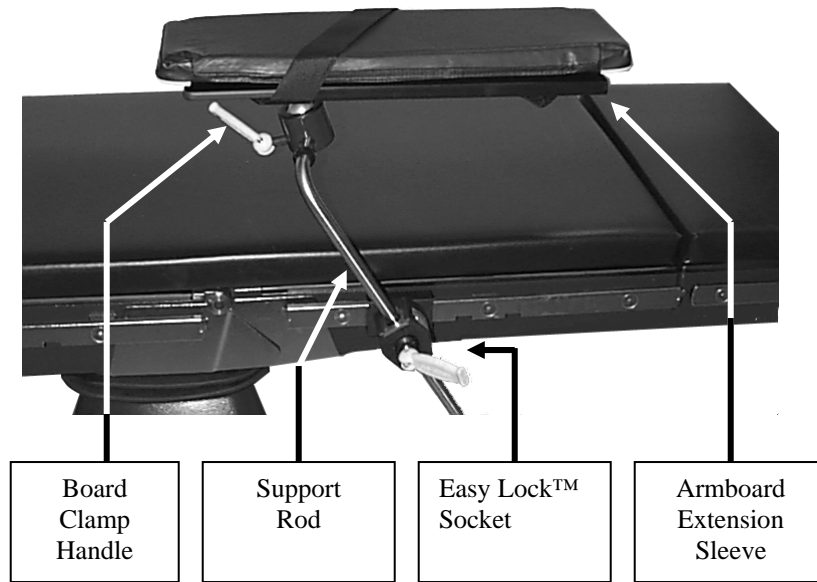

**CAUTION:** Do not exceed 500 pound patient weight limit.

Equipment set up and use:

- **To Install:** Insert the Support Rod into an Easy Lock™ Socket clamp. Tighten the Accessory Lock Knob securely to prevent the Support Rod from rotating.
- **To Adjust Length:** Pull the Armboard Extension Sleeve
- **To Adjust Radial Angulation:** Unlock the Board Clamp Handle and move the armboard to the desired location. Retighten the Board Clamp Handle to secure the board in the new position.
- **To Adjust Height:** Vary the amount of the Support Rod that is inserted into the Easy Lock™ Socket clamp.

Equipment uses:

Armrest



Prone Arm Support



Lateral Patient Brace



Neuro Arm Support



Lateral Arm Support



Supine Arm Support



Passionate About Positioning™



**Cleaning and Maintenance:**

1. After each use, clean the device.
2. Clean and disinfect using a quaternary ammonium disinfecting/cleaning solution following the manufacturer's recommendation for achieving low-level disinfection.
3. Read the cleaning product's directions and follow the instructions on the label. Use caution in areas where fluid migration may occur:

Handles

Clamps

4. Wipe device with a clean, dry cloth.
5. Make certain the product is dry prior to reinstalling to avoid damage.



**CAUTION:** Damage may result if product is cleaned with bleach.

**Suggested accessories:**

Easy Lock™ Socket Clamps:

A-40022 (US)

A-40023 (Eur)

A-40024 (UK)

A-40030 (JP)

Pads:

O-AMA12X1.9S (Standard, 1" (2.5cm) Armboard Pad)

O-AMA97X1.0S (Clamshell, 1" (2.5cm) Armboard Pad (Wrap))