

Operation Manual for Control Panel XMT-308

Warm & Cozy Blanket Warmer

1. Pls refer to "User's Manual" first. All of Warm & Cozy blanket warmer include "User's Manual" at the carton box.

2. The temperature control

1) The factory set the temperature at 130 F (Fahrenheit) +- 10%

After turn-on the power, pls wait about 30 minutes for it to fully heat up.

The temperature will be moved from 115 F to 145 F.

2) For re-set the temperature

a) PRESS & HOLD **the UP or DOWN arrow button** for about 2-3 seconds.

Then select the desired temperature setting by pushing the up or down arrow button to increase or to decrease the temperature.

b) Once a desired temperature has been selected, push **SET button** to save the temperature.

3. Re-Set the initial value (to make the factory re-set value)

1) Pls refer to attached **XMT-308 Temperature Control Instruction Manual**

(at the 2nd page, pls refer to "First Menu" for ex-factory value.)

2) PRESS the SET button for 2-3 seconds.

Window display **INDEX SYMBOL** (ex, AL, SC, P, I, d, t, hy, At, Loc)

and **INDEX VALUE** (ex, 200, 0, 15.0, 240, 30, 10, 1.0, OFF, 0).

- 3) Re-Set the initial value
- a) Press **the SET button** for 2-3 seconds.
 - b) INDEX symbol **"AL"** will be display (High Limit Alarm)
 - ⇒ By using UP or Down button to make INDEX value with **"200"** and Press "SET" button one more time to save.
 - ⇒ Automatically the display will be next Index Symbol of **"SC"**.
 - c) INDEX symbol **"SC"** → INDEX value **"0"**
 - d) INDEX symbol **"P"** → INDEX value **"15.0"**
 - e) INDEX symbol **"I"** → INDEX value **"240"**
 - f) INDEX symbol **"d"** → INDEX Value **"30"**
 - g) INDEX symbol **"t"** → INDEX value **"10"**
 - h) INDEX symbol **"HY"** → INDEX value **"1.0"**
 - i) INDEX symbol **"At"** → INDEX value **"OFF"**
 - j) INDEX Symbol **"Loc"** → INDEX value **"0"**
 - k) After 2-3 seconds, the current temperature will be displayed.

Thks & B/Rdgs.

Point A Tech.