



“Accufit” MRI Coil Drape Instructions For Model # BSM1007-01 and # BSM1008-01

The “AccuFit” MRI Coil drape is designed to protect your equipment and patients from any oils, dirt or contact skin issues. This coil drape is compatible with the coils listed and may fit other models:

Siemens 1.5T Avanto/Espree 7 Channel
 Philips 1.5T Achieva 7 Channel
 GE 8 Channel coil
 GE 3.0T 7 Channel Breast
 May fit coils other than those listed

Siemens 1.5T Symphony 7 Channel
 Philips 3.0T SENSE
 GE 1.5T Excite 7 Channel
 CP Siemens Breast Array



Special features are designed into this drape. Occasionally due to lesion accessibility, the coil pad is removed. To accommodate this situation the Coil cover is lined with extra fabric for patient comfort.

1. Open the package. The top of the drape will align the round holes to match the coil holes.



2. Pull elastic around all edges for a secure fit.

NOTE:

The inner padding allows the coil pad to be removed and the AccuFit™ Coil drape to be utilized. The padding faces down next to the coil

