

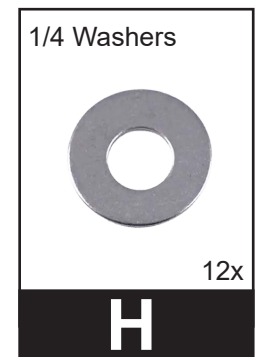
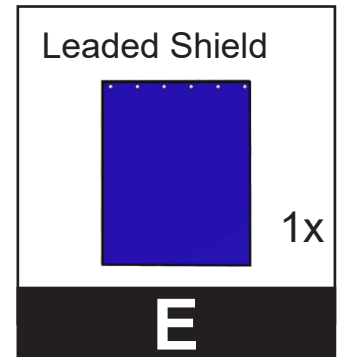
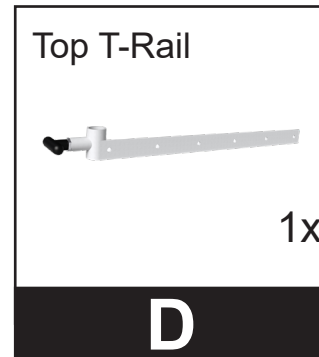


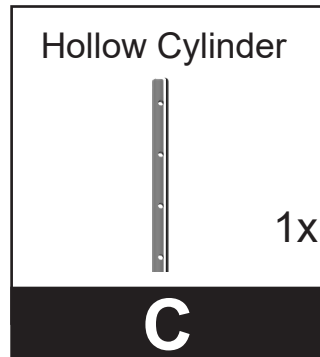

# ASSEMBLY INSTRUCTIONS

## DELUXE MOBILE LEAD SHIELD ON T-BASE

COPYRIGHT ©2025. ALL RIGHTS RESERVED



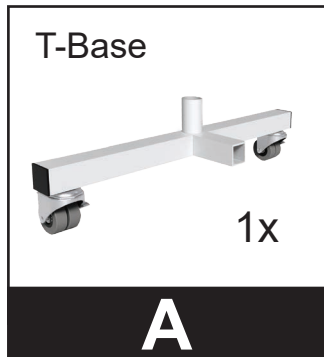


**ASSEMBLY INSTRUCTIONS**  
MOBILE LEAD PORTA SHIELD

**STEP 1**

1. Get the T-Base **A** and the Straight One Wheel Base **B**.
2. Attach the T-Base **A** to Straight One Wheel Base **B**. The beam will go over the smaller square rod.
3. Tighten Allen Set Screws into place.

**PARTS NEEDED**

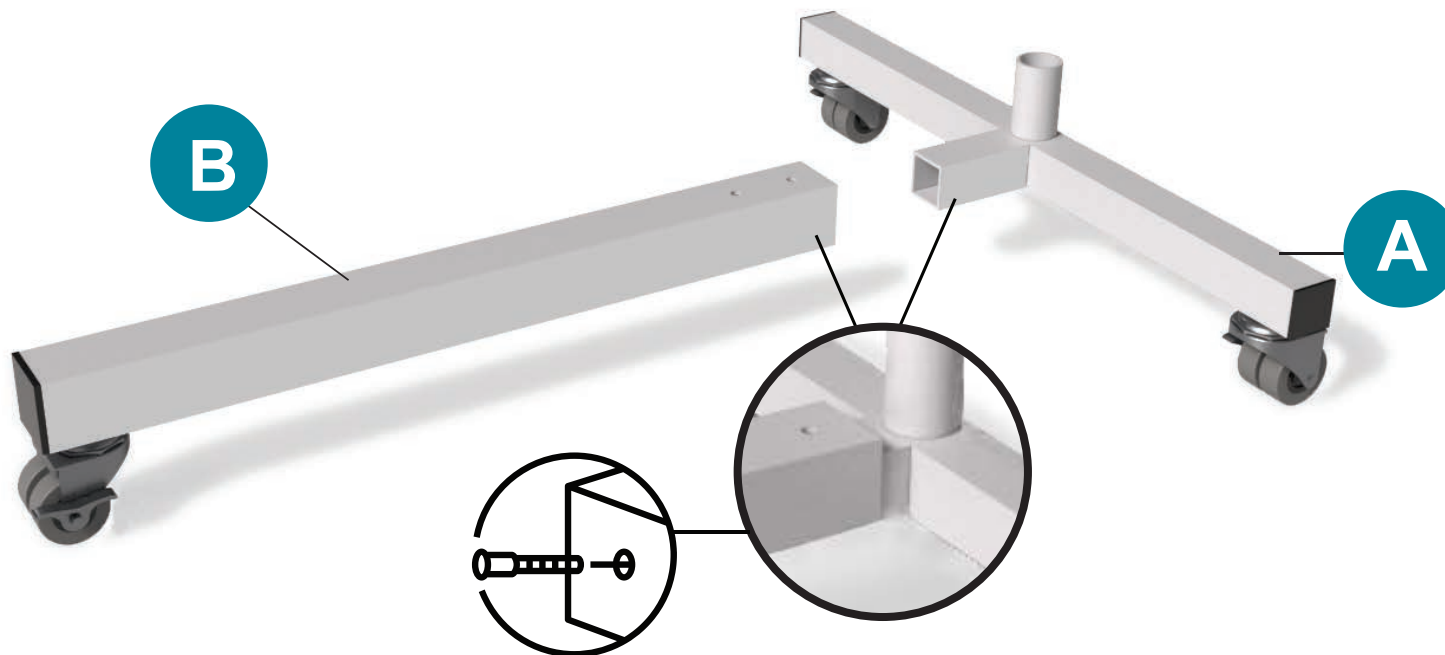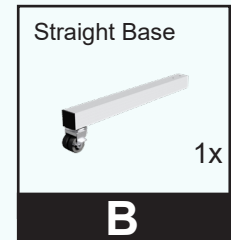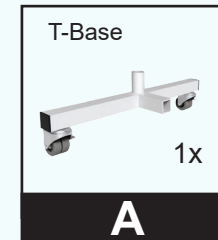



**T-BASE ASSEMBLY**


**ASSEMBLY INSTRUCTIONS**  
MOBILE LEAD PORTA SHIELD

**STEP 2**

4. Insert the Hollow Cylinder **C** into the finished T-Base **A + B**.
5. Make sure the Hollow Cylinder **C** is straight and the holes are facing outward.
6. Secure into place.

**PARTS NEEDED**

T-Base + Straight Base  1x	Hollow Cylinder  1x
<b>A+B</b>	<b>C</b>

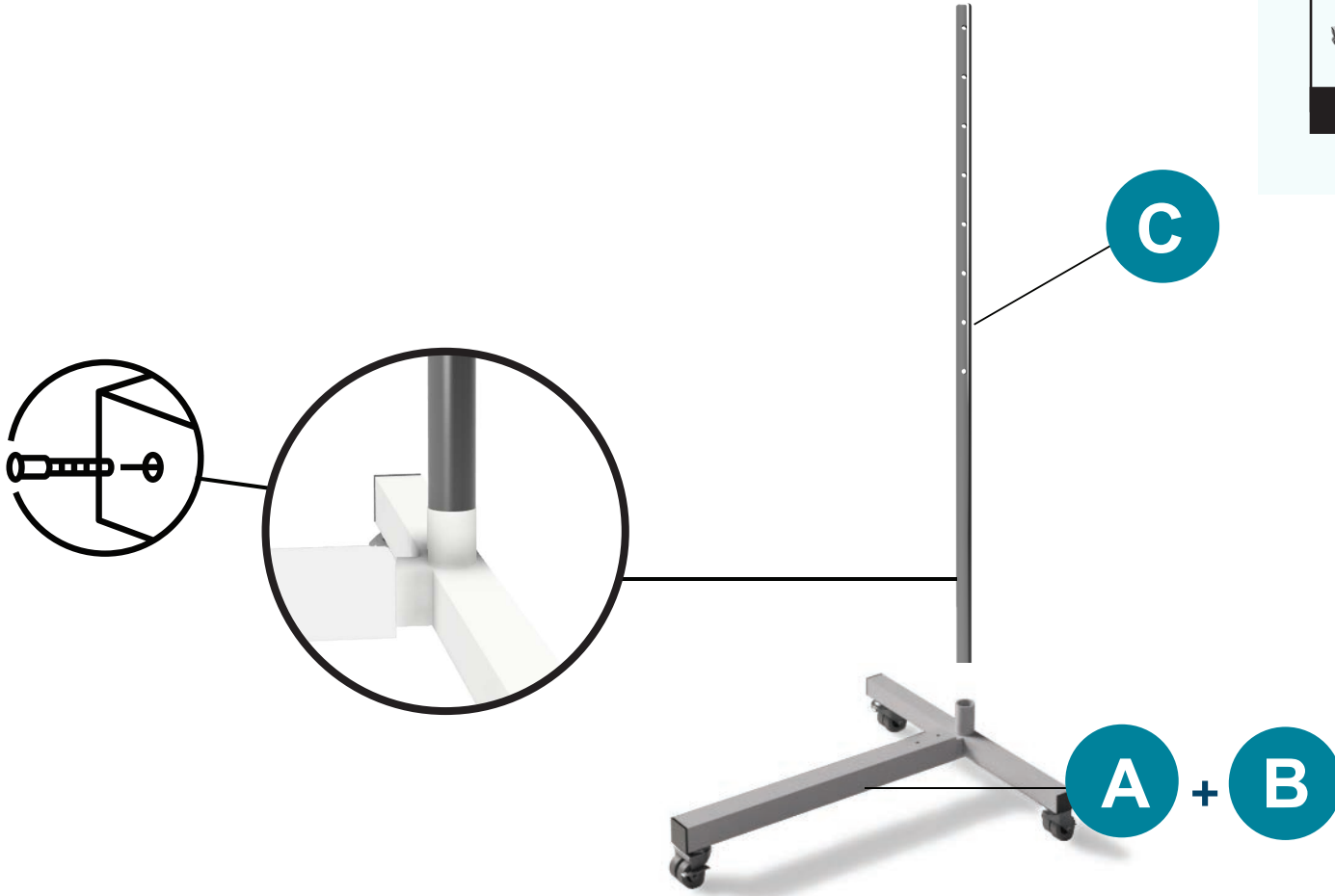


**HOLLOW CYLINDER + T BASE**


**ASSEMBLY INSTRUCTIONS**  
MOBILE LEAD PORTA SHIELD

**STEP 3**

7. Attach the Top T-Rail **D** into the **assembled Hollow Cylinder**.
8. Pull the spring loaded black handle outwards.
9. Align the hole of the Top T-Rail **D** with the Hollow Cylinder **C**.

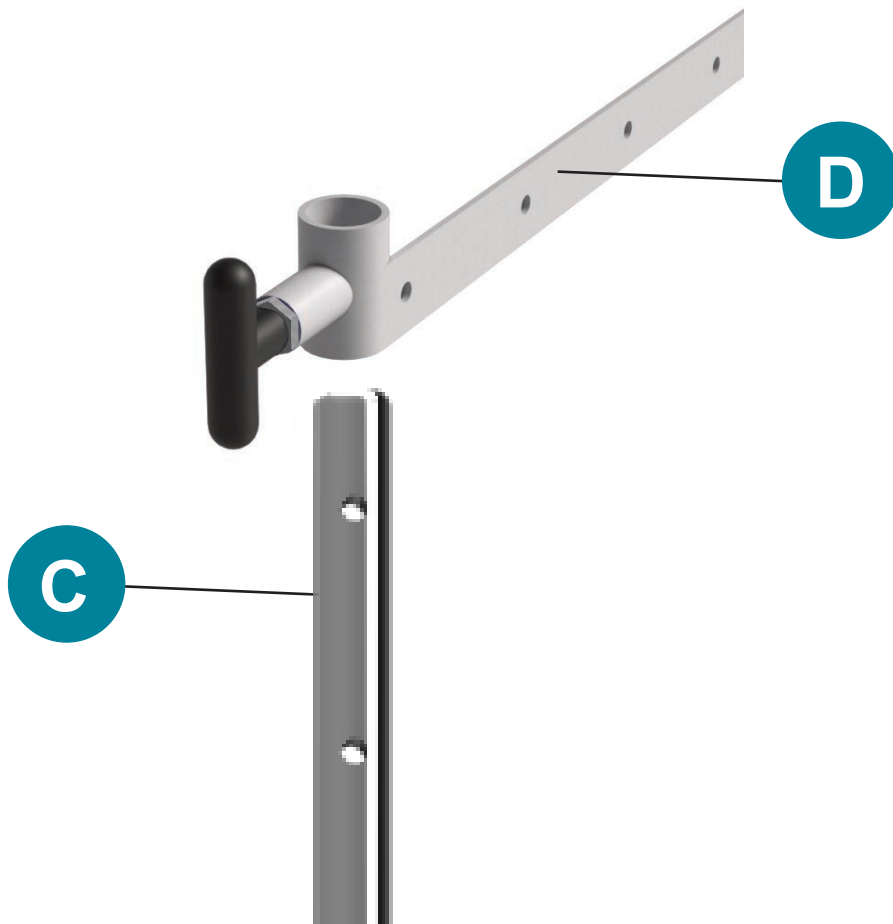

**PARTS NEEDED**

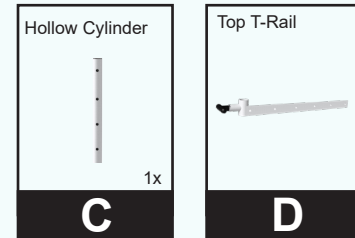


**TOP TRAIL**

**ASSEMBLY INSTRUCTIONS**  
MOBILE LEAD PORTA SHIELD


**STEP 4**

- Align the Leaded Shield **E** with the Top T-Rail **D** holes.
- To secure the Leaded Shield **E** in place use the hardware in the following order:


**G** + **H** + **E** + **D** + **H** + **F**

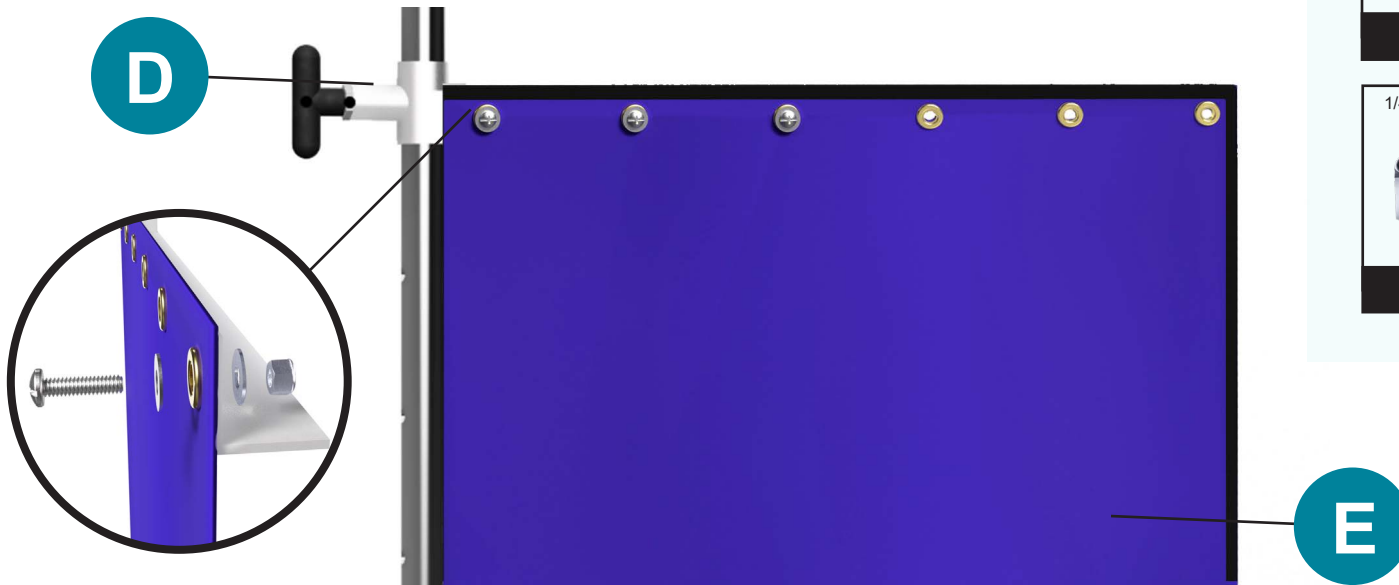


**PARTS NEEDED**

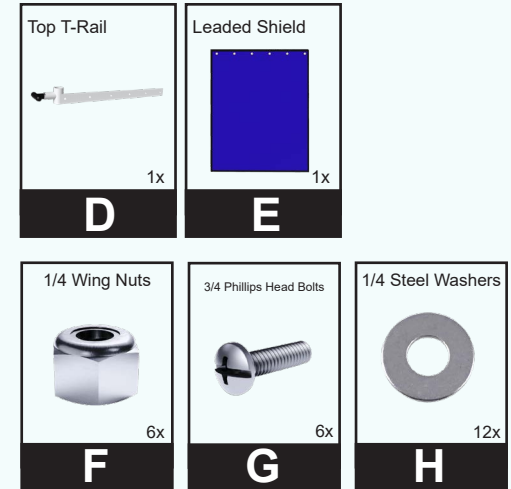

**FRONT**

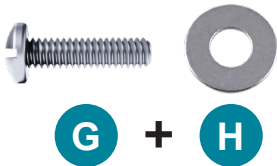

**BACK**

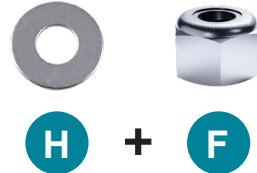

**LEADED SHIELD**


**ASSEMBLY INSTRUCTIONS**  
MOBILE LEAD PORTA SHIELD

**STEP 5**

12. Pull plunge pin handle out.
13. Slide arm up/down to desired height.

**ATTENTION:**

Because of the weight of the shield, it is recommended that a second person help to move the arm by supporting the weight at the far end of the arm.

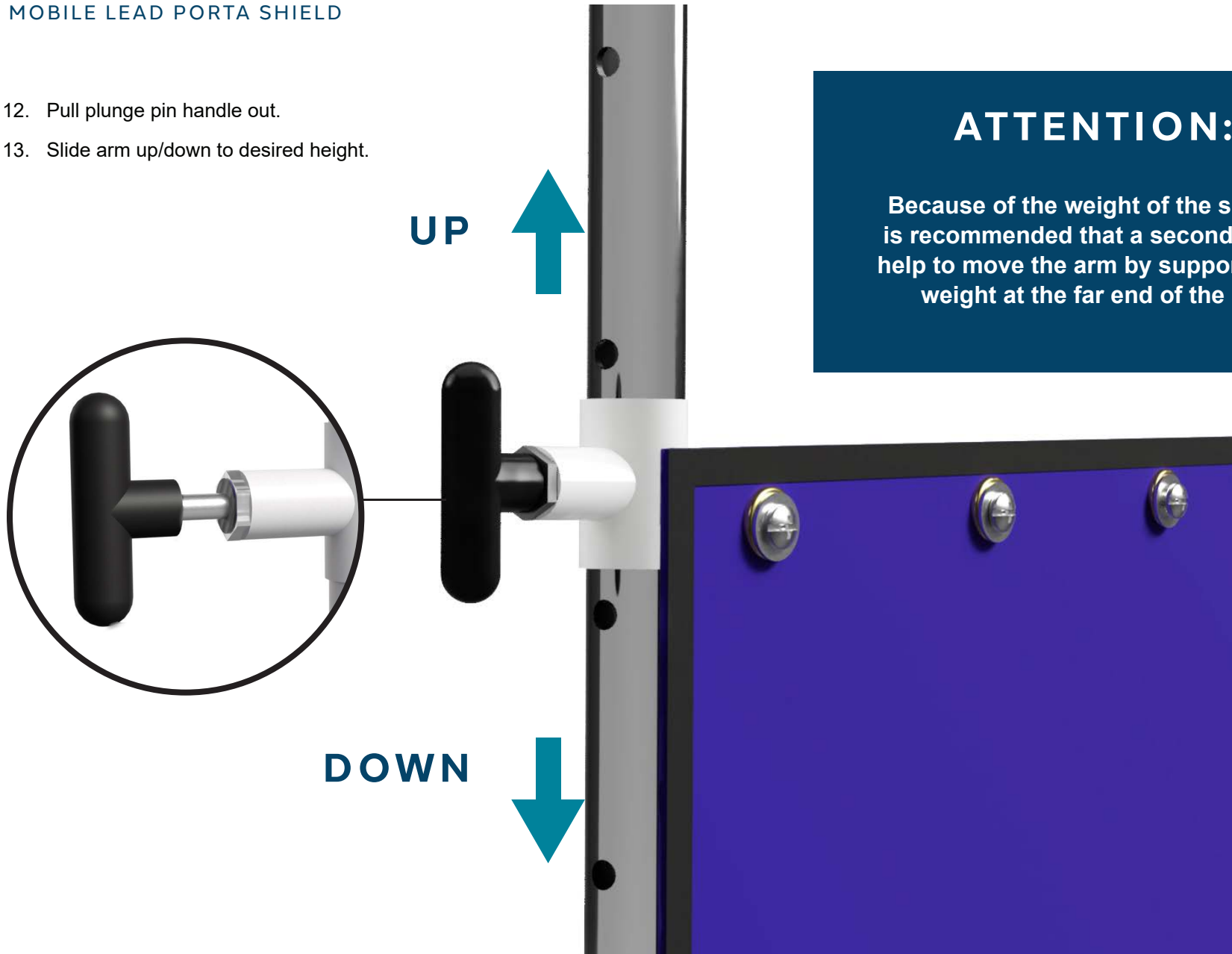


**ADJUSTING PRODUCT HEIGHT**

**ASSEMBLY INSTRUCTIONS**  
MOBILE LEAD PORTA SHIELD

**STEP 6**

14. Tighten all loose screws and bolts.

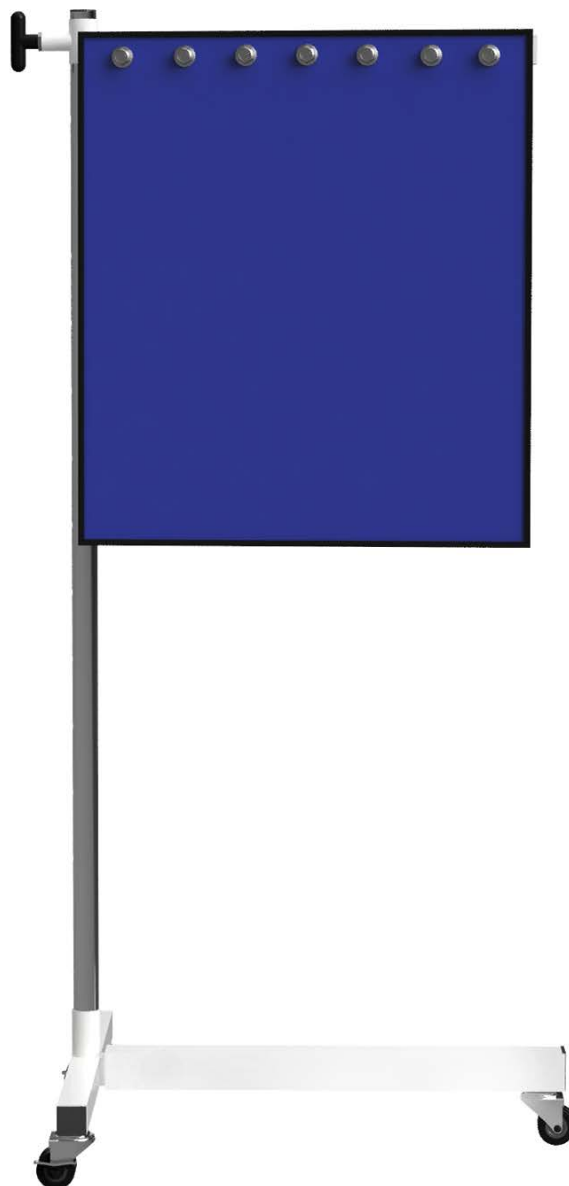

**FINAL PRODUCT**